

DOCA5983 U-Bolt Installation, PS

Tool Free Installation

After removing your new attachment from the packaging, remove the knurled knobs and lock washers. (Shown Below)

Slide the U-bolt out of the saddle, separating the two parts.


Choose the desired height of the attachment.

Push the saddle of the U-bolt clamp up against the Pedestal Stand Pole.

Re-Insert the U-Bolt back into the saddle, this will wrap the U-bolt around the pole. The fit should resemble the illustration below.

Apply a lock washer over one of the threads protruding the saddle. Thread a knurled knob onto the thread. Do Not Tighten. Repeat this process on each of the U-bolt threads. (Depending on which style attachment you have, it will be 2 or 4 threads.)

Adjust the unit to the height and/or orientation you would like on the pole. Hand tighten the knurled knobs and the attachment is ready for use.

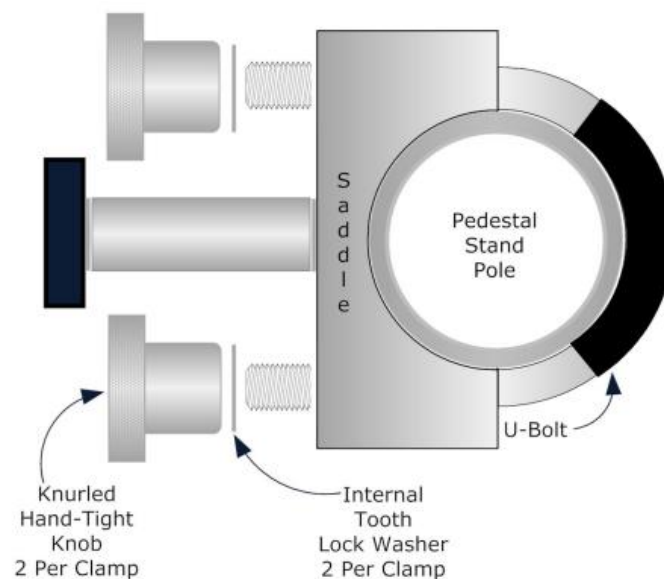

Available Pedestal Stand Attachments

000A6179	Mayo Instrument Side Tray
000A6185	IV Pole Attachment
000A6186	E-Cylinder Tank Attachment
000A6189	Basket Attachment #1, 10.5" x 5" x 13"
000A6190	Basket Attachment #2, 9" x 6" x 11.5"
000A6191	Basket Attachment #3, BP Cuff
000A6192	O2 DISS Dual Interface Attachment
000A6193	Power Strip Attachment

Hallowell EMC
35 Downing Industrial Park
Pittsfield, MA 01201
Ph: (413) 445-4263
Fax: (413) 496-9254
www.hallowell.com